



**L&R Manufacturing Company**

577 Elm Street, P.O. Box 607, Kearny, New Jersey 07032-0607 USA

Phone: 201-991-5330, Fax: 201-991-5870

E-mail: [info@LRultrasonics.com](mailto:info@LRultrasonics.com), [www.LRultrasonics.com](http://www.LRultrasonics.com)

## **PODIATRY**

Surgical instruments (scalpels, blades, burs, forceps, clippers, hemostats, rasps, even jet injectors) can be cleaned quickly, safely, economically and thoroughly with Ultrasonic Cleaning Systems from L&R. Simply-

1. Place instruments in Basket (hinges open) and immerse in tank filled with properly diluted General Medical Cleaner Concentrate (1:10).
2. Cavitate 5 minutes, remove, rinse thoroughly in water.
3. Dry instruments and sterilize.

Note: Jet injectors should be cleaned in a 600 ML Beaker. Place Beaker into Positioning Cover and cavitate for 3 minutes.

### **EQUIPMENT NEEDED:**

Machines, Accessories\*: 2014, Q140 or S140 Ultrasonic Cleaning System with Timer, Optional Heater\*\*, Basket, Auxiliary Pan, Positioning Cover, 600 ML Beakers.

Solution: General Medical Cleaner Concentrate (Product Code 149).

\*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.

\*\*Heater not available on Model 2014.