



L&R Manufacturing Company

577 Elm Street, P.O. Box 607, Kearny, New Jersey 07032-0607 USA

Phone: 201-991-5330, Fax: 201-991-5870

E-mail: info@LRultrasonics.com, www.LRultrasonics.com

VETERINARY HOSPITALS, ANIMAL CLINICS

Rather than hand cleaning and dulling sharp cutting edges and wasting time cleaning hard-to-get-at places in pipettes, curettes, scalers, trimmers and probes, modern 3-dimensional cleaning is available with L&R Ultrasonic Cleaning Systems. Simply-

1. Place Basket containing instruments into tank filled with properly diluted General Medical Cleaner Concentrate (see label for dilution rates). Hinged instruments should be opened.
2. Cavitate 5 minutes.
3. Remove Basket. Rinse thoroughly in water.
4. Sterilize per normal procedure.

EQUIPMENT NEEDED:

Machines, Accessories*: 2014, Q140, S140, Q210 or S200 Ultrasonic Cleaning System with Timer, Basket, Auxiliary Pan.

Solution: General Medical Cleaner Concentrate (Product Code 149).

*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.

APPLICATION BULLETIN #52